Protac Kuglepuden™

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Protac Ball Cushion™

GB - Directions for use

Protac Kugelkissen™

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Protac pallotyyny™

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Protac Ball Cushion™

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Protac Ball Cushion™

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Protac Bolldynan™

SE - Bruksanvisning

Almohadilla Protac Ball Cushion™

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Protac Ball Cushion™

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Protac Kuglepuden™

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GB - Directions for use

Protac Ball Cushion™

You have selected a quality Protac product. Protac guarantees that you have purchased an original, Danish designed product developed in consultation with experts.

Protac has developed, produced and sold sensory-stimulating aids to the healthcare sector in Denmark, the rest of Europe and Australia since 1994.

Protac products are developed on the basis of theories on sensory integration, and are supported by extensive research, documentation and case studies.

Protac focuses strongly on protection of the environment and safety during its production process. Protac's plastic balls are TÜV-approved. The polyester textile is Oeko-Tex standard 100 certified.

For more details, go to www.protac.dk

GB

Labelling

Our products are labelled according to legislation concerning medical equipment class I.

The label with details of manufacturer, raw materials and washing instructions is sewn on the side of the cushion to avoid irritating the user.

Key to label symbols: please refer to **Key to symbols**

General precautions - Safety

- ⚠ • The cushion **must** be securely fastened to a chair
- ⚠ • Please **be aware** of how the user reacts
- ⚠ • **Not suitable** for wheelchair-users
- ⚠ • **Not for unsupervised use** by users with paralysis and poor sensitivity.
- ⚠ • **Must not** be used near an open flame

Product purpose and areas of use

Sensory integration is the idea behind the development of all Protac products. The theory originated in the 1970s in the USA, and was developed by A. Jean Ayres, an occupational therapist and psychologist. The theory is based on how the brain processes sensory impressions, and how the various sensory stimuli can be influenced via applying weight and pressure to the tactile sense (the sense of touch) in the skin and the proprioceptive sense (the sense of movement) in muscles and joints. The pressure and movement of the balls act on these senses in particular.

The Protac Ball Cushion™ was developed to help sufferers of function impairment.

Children, young and old people can benefit from the cushion:

- Brain damage, including spasticity, Huntington's Chorea, Parkinson's Disease and other ailments
- ADHD and autism
- Psychiatric diseases, such as dementia, schizophrenia, depression, anxiety, anorexia etc.
- Development problems, sensory problems, deaf/blind, etc.
- Stress

The cushion is intended for patients with brain damage suffering from spasticity and sensitivity impairment, to sit on. The weight and pressure of the balls help suppress spasticity to give the patient enhanced body awareness. It can also be used to train balance with. Sufferers of ADHD and autism can use the Protac Ball Cushion™ to sit on in school or the home when unconcentrated. When the user moves on the cushion, the pressure of the balls stimulates muscles and joints, helping improve body posture.

Psychiatric patients often suffer from muscular and psychological unrest. The cushion can be used to enhance concentration and help calm the patient. The cushion can be used for meeting and dining.

The Protac Ball Cushion™ is used for sufferers of a range of development problems, sensory problems, deaf/blindness and to create enhanced sensory stimuli in the body.

It is used for sufferers of stress to provide calmness, comfort and better body awareness.

The Protac Ball Cushion™ can be used in institutions, sheltered housing, day centres, hospitals, patient lounges and private homes. The cushion is used in schools in the classroom for instance, and at home, can be used on dining or office chairs.

Using the product

The cushion must be placed on a hard surface, e.g. a classroom stool or one-piece moulded chair to gain full benefit of sensory stimulation.

The cushion must be securely fastened to a chair.

We recommend ensuring that the child has something to rest his/her feet on, e.g. the floor, a foot stool or similar.

The cushion can be used on a daily basis in normal use. Because the child constantly moves on the cushion, sitting on the balls does not feel uncomfortable.

Product service life

The product has an expected service life of 5 years.

This is based on correct use, cleaning, washing, service and maintenance.

Products used beyond the aforementioned service life should be assessed for further use.

Responsibility for using the product beyond the stated service life rests with the user.

Cleaning

Protac Ball Cushion™ can be washed. The regularity and need for washing depends on use. See below for washing and cleaning instructions.

Protac Ball Cushion™

- can be machine washed in a min. 6 kg capacity washing machine with washing powder with no bleaching agents - see also the washing instructions on the cushion.

To remove spots using a spot remover, always try first on an area which is not visible to test for damage or discolouring. Protac does not recommend the use of spot remover.

Protac cannot be held liable for incorrect washing and maintenance.

Maintenance

We recommend airing the cushion regularly.

Transport and storage

The cushion is ready for use.

It is supplied packed in a protective plastic bag. If you are not using the cushion, store it carefully in its case to protect from damp and dirt.

Disposal

The Cushion can be broken down into the following components for disposal: Plastic and textile.

The cushion can be disposed of as combustible refuse.

Always dispose of refuse according to local rules.

Guarantee

2 year guarantee against manufacturing defects.

The guarantee will be invalid in the event of the following:

- over-use
- misuse
- non-maintenance or incorrect maintenance

Traceability: The month and year of manufacturer are marked on the washing instructions label.

Key to symbols

Example only. Always follow the recommended care instructions on the label found inside the product

Washing bowl: Maximum washing temperature (see care label)

Warning:
See under **Safety**

Triangle:
Do not bleach

User information

Square with a circle:
tumble drying possible
(see care label)

For indoor use

CE approval according to Directive concerning medical devices 93/42/EC Class I

Iron:
Do not iron

Manufacturer

Circle:
Do not dry clean

Product serial number (see care label)

Technical specifications

Dimensions:

Protac Ball Cushion™ for classroom chair:	40x40 cm
Protac Ball Cushion™ for baby's high chair:	29x45 cm
Protac Ball Cushion™ for kindergarten chair:	30x35 cm
Protac Ball Cushion™ for stool:	30x30 cm

Standards: Comply with general requirements for aids acc. to the following harmonized standards:

Clinical evaluation: EN 12182:2012

Risk analysis: DS/EN ISO 14971

Fire test: EN 1021-1: 2006 and EN 1021 - 2: 2014

GB

Service information

No other balls can be used in a Protac Ball Cushion™ than those used by the manufacturer.

Guide

- to daily inspection of the Protac Ball Cushion™ before use

No.	Check	Remarks
1	<ul style="list-style-type: none">All parts are in place, no loose parts	
2	<ul style="list-style-type: none">No sign of holes, damage to the textile or seams	

Protac A/S

Niels Bohrs Vej 31 D, Stilling
DK-8660 Skanderborg
Tel.: 86 19 41 03
e-mail: protac@protac.dk

Dealer list: Please refer to: www.protac.dk/kontakt

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